Instructions for Visual Otoscopes


Type : AGC-450

Product Profile:

This product is a digital high-definition visual otoscope, mainly used for internal ear inspection, can assist ear cleaning and ear care, etc. This product is plug and play, does not need to install any software, simple operation, stable performance!

Operation Instructions: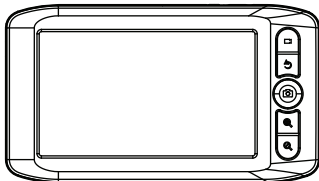


 **LED adjusting button:** level 0-4 cycle; level 0 LED is completely extinguished; level 4 LED is the brightest; short press one level at a time, cycle in turn!

 **Photo or video button:** short press to take photo; long press to video; as the up-flip button in menu mode!


OK Menu button: long press to enter menu mode; as OK button in menu mode!

 **Enlargement button:** short press enlargement; maximum 2 times enlargement; short press one level at a time; circulate between 1.0 times, 1.4 times, 1.6 times, 2.0 times; as the down-flip button in menu mode!

 **Rotate button:** short press for photo rotation; long press for file browsing mode!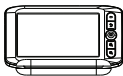

Application Scenarios:

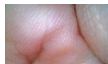Product parameters:

Camera parameters:

Photo sensor: CMOS sensor

Effective Pixel: 1280*720

Focal length: 20mm

Visual angle: 60°

Lens size: 3.9mm

LED: 6LED, level 0-4 cycle

Frame rate: 30FPS

Photo format: JPG

Video format: AVI

Line length: 150cm

Waterproof grade: IP67(Camera)

Host parameters:

Screen: 4.3-inch IPS screen

Screen resolution: 480*272

Battery capacity: 1900mA

Storage capacity: 32G (Max)

Working Current: 400mA

Working voltage: 5V

Standby current: 40mA

Maximum power: 2W

Product list:

4.3-inch host	1pc
Camera	1pc
Supporting base	1pc
Charging line	1pc
Ear cleaner	2pc
Cotton swab	2pc
Mirror cloth	1pc

Common problem:

Question1: How do I read the files in the TF card directly through the computer?

Answer: Please plug in the USB data line under shutdown condition and then turn on. The data of TF card can be read directly through the computer.

Question2: Why does the new TF card indicate that the TF card is full?

Answer: Please use the standard FAT32/Class10 TF card. Please format it before using it.

Question3: How to judge whether the battery is full?

Answer: When the machine is in working state, plug in the USB data line, then the system will automatically charge the battery. There will be charging icon indication in the upper right of the screen , the indicator will also turn on the red light, and the red light goes out when full. If charged when the power is off, the indicator turns on red light, and the red light goes out when full!

Question4: Does the proto look blurred?

Answer: First, please check whether there is dirt in the camera. If there is dirt, please wipe the camera clean with a mirror cloth. The effective observation distance of the camera is 10-20 mm. In this range, the object observation can achieve the clearest photo effect.

Question5: The waterproof grade is IP67. Is it waterproof for the whole machine?

Answer: This product is not waterproof for the whole machine. IP67 waterproof parameter identification, refers to that the part of the camera waterproof grade is IP67, does not include joints and host parts!

Points for attention:

- Keep the product clean and avoid direct contact with oil, water and other dangerous goods!
- Please keep the lens clean!
- This product is only suitable for human ear inspection, not as a medical diagnostic equipment!
- Please don't let children touch and operate this product!
- If there is any damage, please do not disassemble it by yourself. Contact the supplier for maintenance service.

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